Technical Specifications



O II	1+	
Overall	dime	ensions

- Mattress platform
- Height
- Side rail height
- Side rail length
- Backrest angle
- Kneerest angle

Casters

- Bed weight Safe working load

2260 mm × 1010 mm 2000 mm × 810 mm

440 mm 410 mm 1620 mm

125 mm

0 - 700 0 - 35°

90 kg 220 kg

Technical and Quality Assurance Standards

IEC 60601-2-52 ISO 13485 ISO 14971

Positions

Piyatech Sdn. Bhd.

www.piyatech.com email: info@piyatech.com



B3-U5-7, Solaris Dutamas, Jalan Dutamas 1,

50480 Kuala Lumpur, Malaysia Tel: (+60 3) 62113120, 62112120















Piyalech

General Bed 25000MC1 Type 2











Features





Side RailsLocking Mechanism

Pushing up the red lever on either side of the bed engages the release mechanism allowing the side rails to fold down.

Side Rails

The parallel side rails of the 35000MC1 offer a unique and stylish design produced from a combination of blow molded plastic and stee I bars to provide a protective height and length. The side rails adhere to the highest standards of safety to prevent patient falls and entrapment.



Bed Ends

The seamless blow-molded bed ends are lightweight, durable, and easy to clean. The bed ends are of high quality and compatible with steam cleaning. To have better access to the patient's head in emergency cases, the caregiver can easily remove the bed ends by grasping the hand holds and lifting them up in one single step. Moreover, the bed ends have integrated steel fittings which make them extremely strong so the nursing staff can move the bed by holding and pushing them without worries.



Mattress Platforms

The mattress platforms exhibit exceptional qualities, including smooth surfaces, high durability, and effortless removal for easy cleaning and disinfection. Additionally, the mattress platforms are designed to be compatible with steam cleaning, ensuring thorough and effective sanitation.



Foldable Cranks

Back and knee positions can be adjusted by two foldable cranks located at foot end of the bed.



Casters

125 mm casters run extremely smoothly; the quality that makes it significantly easy for the nursing staff to transport the bed in the medical centers. Having diagonal locks, the casters can be locked to prevent the bed from unintended movement.



Urine Bag Holder

Beds are equipped with a urine bag holder which can be easily installed on either side of the bed.



Bumpers

Innovative rotary bumpers which are located on four corners of beds can prevent to a large extent the damages that may cause collision of the bed to the surroundings during transportation.

General Bed 25000MC1 Type 1





