

General Bed
35000MC1
Type 2




General Bed 35000MC1 Type 2

Technical Specifications



Overall dimensions	2260 mm × 1010 mm
Mattress platform	2000 mm × 810 mm
Hi-Low	~770 mm - ~480 mm
Side rail height	410 mm
Side rail length	1620 mm
Back rest angle	0 - 70°
Knee rest angle	0 - 35°
Bed weight	135 kg
Safe working load	220 kg
Casters	125 mm

Technical and Quality Assurance Standards
 IEC 60601-2-52
 ISO 13485
 ISO 14971

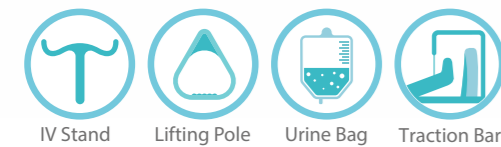

Features



Safety Side Rail Board

The parallel safety side rail board of the 35000MC1 offer a unique and stylish design produced from a combination of blow molded plastic and steel bars to provide a protective height and length. The safety side rail board adhere to the highest standards of safety to prevent patient falls and entrapment.



Safety Side Locking Mechanism

Pushing up the red lever on either side of the bed engages the release mechanism allowing the safety side rail board to fold down.



Foldable Cranks

Back, knee, and hi-low positions can be adjusted by three foldable cranks located at foot end of the bed.

General Bed 35000MC1 Type 2




Bed Ends

The seamless blow-molded bed ends are lightweight, durable, and easy to clean. The bed ends are of high quality and compatible with steam cleaning. To have better access to the patient's head in emergency cases, the caregiver can easily remove the bed ends by grasping the hand holds and lifting them up in one single step. Moreover, the bed ends have integrated steel fittings which make them extremely strong so the nursing staff can move the bed by holding and pushing them without worries.



Casters

125 mm anti-static casters run extremely smoothly; the quality that makes it significantly easy for the nursing staff to transport the bed in the medical centers. Having diagonal locks, the casters can be locked to prevent the bed from unintended movement. (150 mm casters are available as an option)

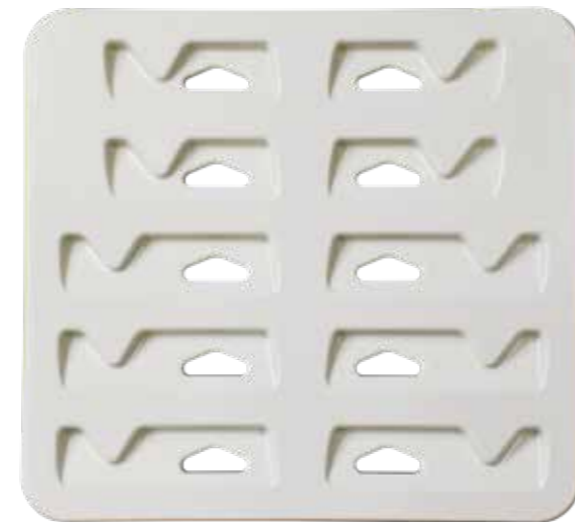

Mattress Platforms

The injection-molded mattress platforms exhibit exceptional qualities, including smooth surfaces, high durability, and effortless removal for easy cleaning and disinfection. Additionally, the mattress platforms are designed to be compatible with steam cleaning, ensuring thorough and effective sanitation.



IV Pole Holder

Four IV Pole Holders are Located on the head and foot end.



Urine Bag Holder

Beds are equipped with a urine bag holder which can be easily installed on either side of the bed.



Bumpers

Innovative rotary bumpers which are located on four corners of beds can prevent to a large extent the damages that may cause collision of the bed to the surroundings during transportation.

General Bed 35000MC1 Type 1



Safety Side Rail



Monkey Bar Holder



Bed Ends

The seamless blow-molded bed ends that are lightweight, durable, and effortless to clean. These high-quality bed ends are compatible with steam cleaning. For enhanced accessibility to the patient's head during emergency situations, caregivers can conveniently remove the bed ends in a single step by grasping the hand holds and lifting them up. Additionally, the integrated steel fittings within the bed ends provide exceptional rigidity, enabling nursing staff to move the bed with ease and confidence.



Bumpers

The bed is equipped with bumpers positioned on all four corners, effectively mitigating potential damages caused by inadvertent collisions with the surroundings during transportation.