

Patient Trolley

8000T1

Type 2




Patient Trolley 8000T1 Type 2

Technical Specifications



Overall dimensions	2092 mm x 800 mm
Mattress platform	1942 mm x 610 mm
Hi - Low	1030 mm - 680 mm
Side rail height	380 mm
Side rail length	1458 mm
Backrest angle	0 - 85°
Kneerest angle	0 - 30°
Safe working load	220 kg
Casters (Central locking)	200 mm

Technical and Quality Assurance Standards
 BS 5402
 ISO 13485
 ISO 14971

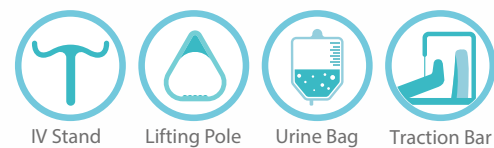

Backrest 0-85°



Kneerest 0-30°



Fowler



Max. height 1030 mm



Min. height 680 mm



Foot Elevation

Features



Safety Side Rails

CPR Bed has a pair of durable, robust, and easy to clean safety side rails that provide a protective height and length to ensure the patient's safety. Both safety side rails are equipped with innovative and easy to use locking mechanisms that allow the rails to be securely locked in an upright position to maximize the patient's safety. Pulling the upward locking handle, the caregiver can easily collapse the side rails.



Safety Side Rail Locking Mechanism

Pushing up the red lever on either side of the bed engages the release mechanism allowing the safety side rails to fold down.



Casters

200 mm anti-static central locking casters run extremely smoothly, the quality that makes it significantly easy for the nursing staff to move the bed. In addition, the foot pedals allow effortless transition between the three modes of the central locking system:

- Safe central locking (down)
- Easy maneuvering and positioning (middle)
- Fast directional movement (up) to provide a smooth and controlled single person transportation of the bed in the medical centers.

Patient Trolley 8000T1 Type 2



Adjustable Grip Handles

Two pairs of adjustable handles at the head and foot end of 8000T1 allow the nursing staff to move the trolley easily in the medical centers.



IV Pole Holder

Two IV pole holders are located in each corner of the head side of trolley.



Mattress Platforms

The trolley's ABS mattress platforms boast many helpful features, like smooth surfaces, high durability, and a lightweight, easily transportable frame enabling quick and painless removal for cleaning and disinfection, including steam cleaning.



Bumpers

Innovative rotary bumpers which are located on four corners of beds can prevent to a large extent the damages that may cause collision of the bed to the surroundings during transportation.



Urine Bag Holder

Urine bag holders can easily install on either side of the trolley in one step.



Oxygen Cylinder Holder

Oxygen cylinder holder is located underneath the trolley.



Hydraulic Cylinder

The lower section of the 8000T1 bed is equipped with a foot pedals on each side, operated by robust and high-quality hydraulic cylinder. This pedal allow for easy adjustment of the trolley height.



Crank

The backrest and Knee rest positions can be adjusted by two cranks that are located at the foot end of the trolley. The cranks can be tucked under the trolley to avoid any collision with the surroundings.



[Option]

The model 8000T1 Type 2 is also offered in 800mm width and different mattress platforms.

